 myergo

Start *your* Journey

by

 UNIVET

Index

01. Quick Guide	05
Components / <i>Componenti • Komponenten • Composants • Componentes</i>	06
How to handle / <i>Come Maneggiarlo • Handhabung • Cómo manipular el producto • Comment le manipuler</i>	07
Settings / <i>Regolazioni • Einstellungen • Ajustes • Réglages</i>	09
02. IFU Instruction for use	11

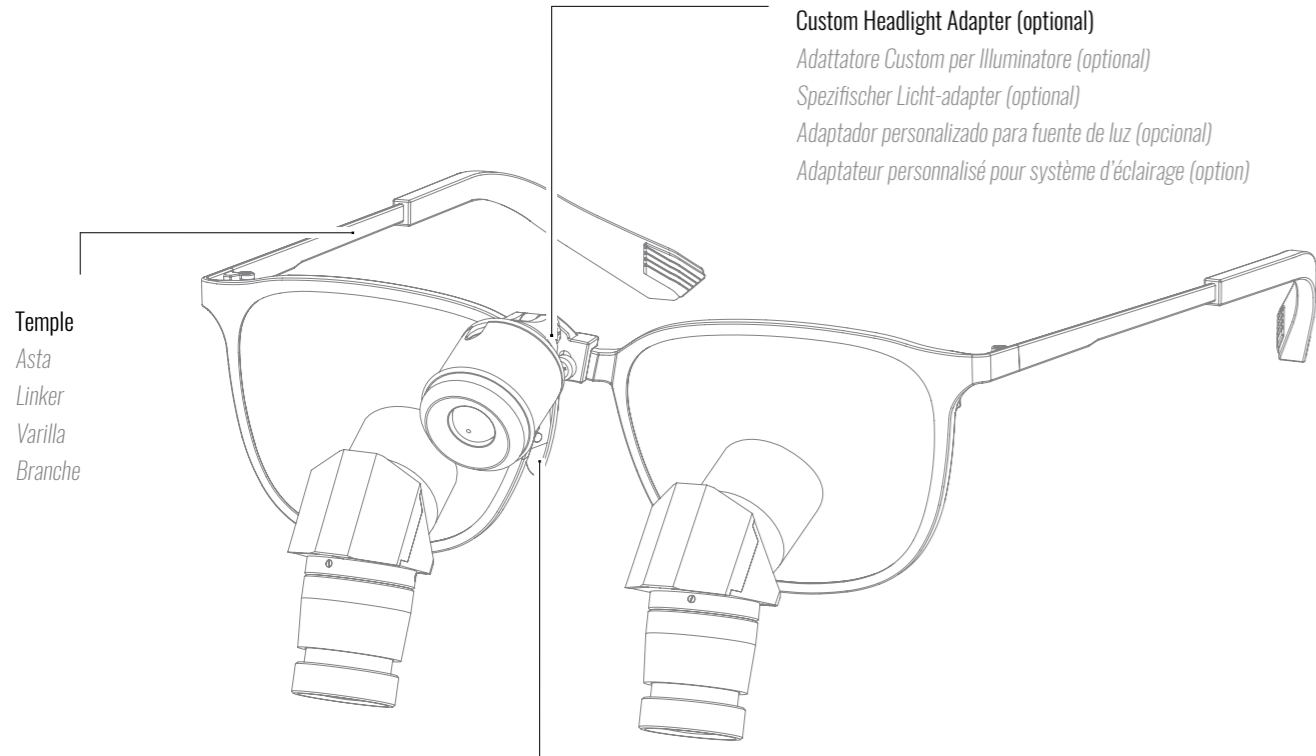
01

Chapter

Quick Guide

5


Components


Temple
Asta
Linker
Varilla
Branche

Custom Headlight Adapter (optional)
Adattatore Custom per Illuminatore (optional)
Spezifischer Licht-adapter (optional)
Adaptador personalizado para fuente de luz (opcional)
Adaptateur personnalisé pour système d'éclairage (option)

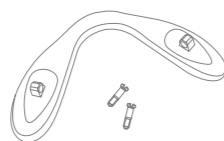Nose pad
Nasello
Nasenplättchen
Puente nasal
Pont nasal

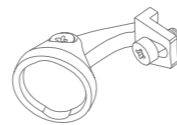Cod. KZA009331
Nose Pad (spare part)
Nasello (Componente di ricambio)
Nasenplättchen (Ersatzteil)
Puente nasal (Pieza de recambio)
Pont nasal (Pièce de rechange)

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How to handle

How to handle

Important! Always handle the loupes by the frame, never by the binoculars; so that the optics are not compromised.

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
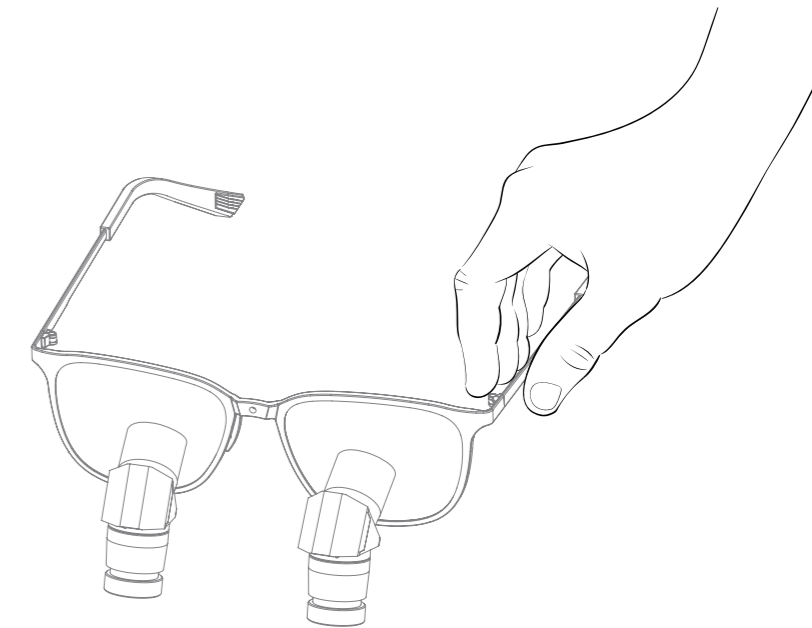
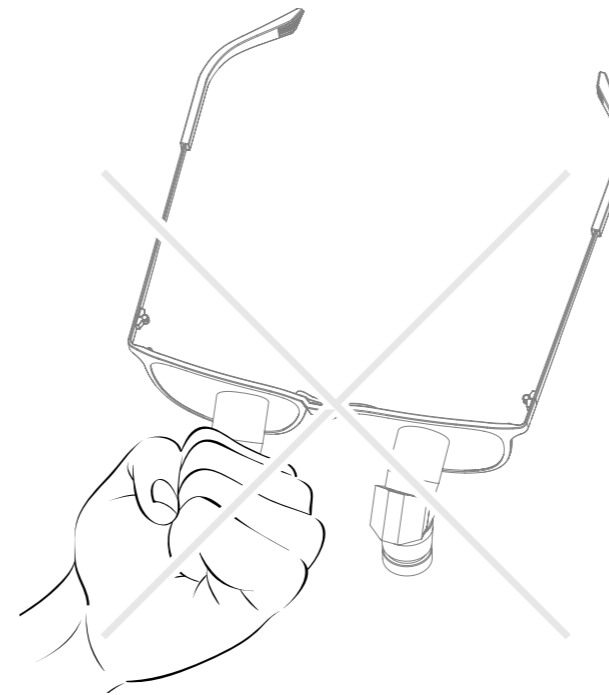
Settings

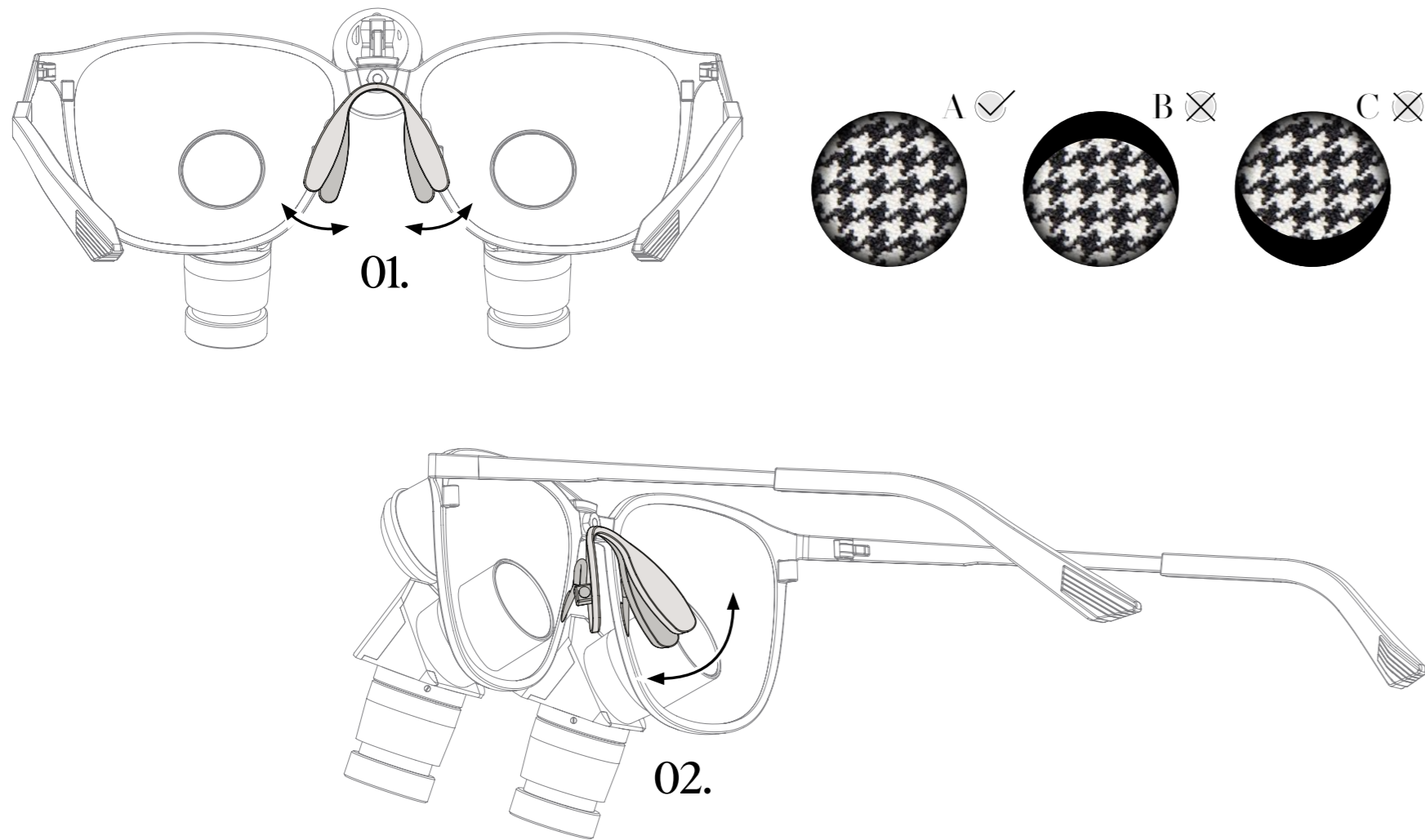Correct Vision

In order to allow correct positioning through the optics, it is advisable to check that the position of the glasses are at the correct height.

Height adjustment

It may be necessary to adjust the nose piece to get the correct view "A".

If the vision appears as in figure "B", it is necessary to tighten both sides of the nose piece (fig. 01). If the vision appears as in figure "C", it is necessary to "flood" them (figure 1).

It is also possible to adjust the inclination of one or both sides of the nose pad (figure 02) to obtain the optimal fit.

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EN | English

 (EN) Instructions for use - MAGNIFYING SYSTEM:
-TTL BINOCULAR LOUPES

Thank you for your trust in us and congratulations on your choice.
Read the following information before using the device.

Univet Loupes Spa
Via G.Prati 87, 25086, Rezzato (BS) Italy
www.univetloupes.com - Email: info@univetloupes.com
Telephone number: +39 030 2499411

DESCRIPTION OF THE COMPONENTS

The user guide for the magnifying system, the list of components and spare parts are indicated in the documents provided along with the device.

SYMBOLS

Below are the symbols (if any) shown on the medical device label.

 In compliance with Regulation 2017/745 on medical devices

 Country of manufacture
 Date of manufacture

 Manufacturer

 Instructions for use

 Medical Device

 Batch number

 Catalogue number

 Serial number

 Caution

INTENDED USE

The device is intended to improve visual acuity in a narrow range of the visual field in medical applications and intended to compensate for a visual defect when equipped with ophthalmic load-bearing lenses.

Intended users: medical professionals.

Intended patients: no limitation except as stated in the “contraindications” section.

DEVICES THAT CAN BE USED TOGETHER

The device can be comfortably attached to all Univet Lighting system models, thus providing an integrated vision system.

CONTRAINDICATIONS

The device (loupes equipped with TTL magnifying system) is not recommended for users suffering from strabismus.

FLIP UP magnifying systems are recommended for operators suffering from strabismus when used along with the corrective spectacles thereof in use.

The device is contraindicated in the event of applications requiring image stability that cannot be achieved with devices mounted on the user’s head.

WARNINGS

 **Read the following warnings carefully before using the device.**

Make sure that it is suitable for the intended use.

Clean the device before use.

Not use the device if it has been damaged but contact your authorised dealer.

Use the supplied components only.

Ensure that the shields (if provided) are correctly fitted, otherwise stop the procedure in progress.

Do not tamper with or disassemble any component of the system.

Do not attempt any repairs.

In the event of a malfunction, stop using the device, contact your authorised dealer.

Avoid contact with heat and cold sources or open flames.

To avoid damaging the eyes, please do not look directly into light sources.

Avoid sudden movements with your head when using the device.

Avoid walking with the device worn.

Do not wear the device over headgear.

Please use the elastic band when using loupes equipped with TTL and FLIP UP magnifying system.

PRECAUTIONS

For any component adjustment and fitting instructions when using for the first time, please see the information provided by the manufacturer. When not in use, store the device in its container.

ADVERSE EFFECTS

No adverse effects detected

SUPPLY MODE

This device is not supplied sterile. This is not a single use device.

STORAGE AND/OR HANDLING

Store the device at the temperatures and humidity indicated in the “Technical Specifications” section. When not in use, store the cleaned device in its original packaging. Handle the device strictly by holding by the temples (in the case of Loupes) or by the bands (in the case of Headgears).

MAINTENANCE

Routine maintenance consists of cleaning the device as described in the “cleaning instructions” section.

Ensure that the components of the device are fitted and adjusted correctly before use.

Keep the original packaging required for return/maintenance shipment and contact your authorised dealer, see the instructions laid down by the warranty.

TRAINING AND QUALIFICATIONS FOR USE

None.

CLEANING

Use a microfiber cloth to clean the lenses.

All the other surfaces can be cleaned with a soft cloth soaked with a gentle detergent solution, then wipe with a dry cloth. Do not clean in an ultrasonic bath. Always avoid immersion of the device in water or other liquids.

DISINFECTION

The choice of the most suitable product and disinfection method varies according to the nature of the contaminant and must be addressed by the user as part of the assessment of the specific risk in the workplace. For a correct choice of disinfectant products compatible with this type of product, we recommend consulting the manufacturer’s website. These products may undergo variations over time for which Univet is not responsible, therefore we always recommend checking the active ingredients of the product before using it. Beware that other types of disinfectants, with different active ingredients, could damage the product. Always check the integrity of the device before using the disinfectant substance. Always follow the instructions on the label of the disinfectant product used. In any case, it is always important that, after exposure, the substance used is completely removed to avoid its accumulation on the device, which over time can cause degradation and / or unexpected breakage. If necessary, use a cloth moistened with water to remove such accumulations, finally proceeding with a complete drying operation, avoiding the permanence of possible accumulations of liquid in the geometry of the device. Always avoid immersion of the device in water or other liquids.

STERILIZATION

The device is not supplied sterile and cannot be sterilized.

DISPOSAL INSTRUCTIONS

The device does not contain toxic or hazardous materials. Disinfect the device before disposal.

In the event of disposal/demolishing, do not dispose of the device in the environment, comply with the relevant national regulations.

HEALTH RISKS

No health risk detected.

WARRANTY

See the instructions laid down by the warranty.

Report any serious accidents involving the device to the manufacturer and the competent authority of the Member state in which the user and/ or the patient resides.

TECHNICAL SPECIFICATIONS

Operating conditions

Room temperature.....from +10°C to +40°C

Relative humidity.....from 10% to 90%

Storage and transportation conditions

Room temperature.....from -10°C to + 50°C

Relative humidity.....from 10% to 90%

Store in a dry place away from heat and cold

CLINICAL BENEFITS

Given that it offers greater visual acuity (magnification) and facilitates a smaller angle of the position of the operator’s neck, the device helps the user to maintain the correct posture during use, hence avoiding the onset of musculoskeletal problems.

No adverse effects for the user.

Univet’s magnifying systems are effective and safe when used as intended.

For languages not included in these use instructions, please see the following link:
www.univetloupes.com/instructions

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All specifications and descriptions are subject to change without notice. All rights reserved. Text, images, graphics are subject to copyright and other protective laws. The content of this publication may not be copied, distributed or modified for commercial purposes without written authorization by Univet Loupes Spa.